


Include AssistDent as a plugin to Dürr Dental's VistaSoft®


This document applies to AssistDent v1.7 and VistaSoft v2.4.

Configure VistaSoft

Within VistaSoft, click the **Configuration** button, select **Interfaces** then scroll down the page to find the option **Edit external applications...**

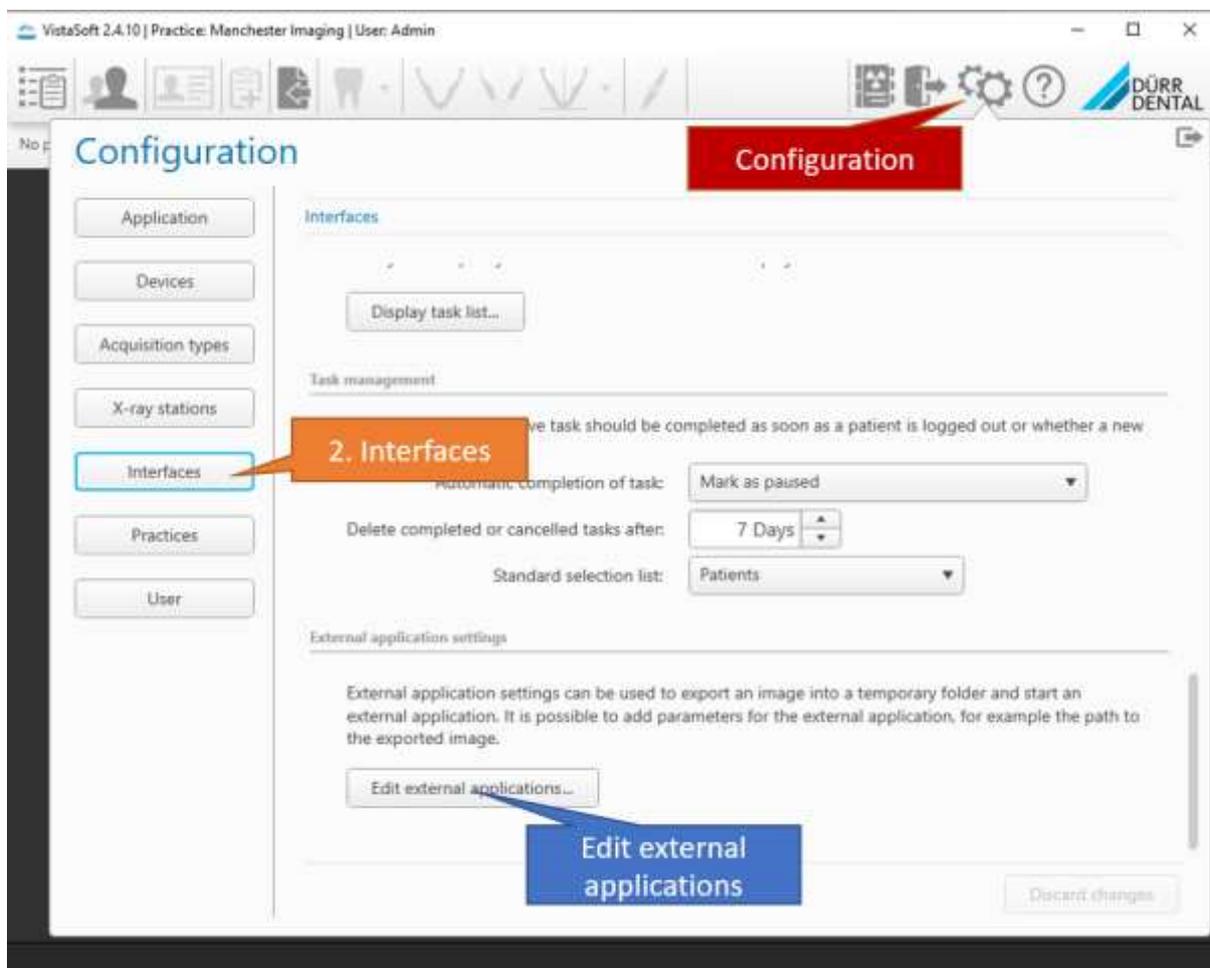

On the Click the external applications page choose the **Create external application** button.

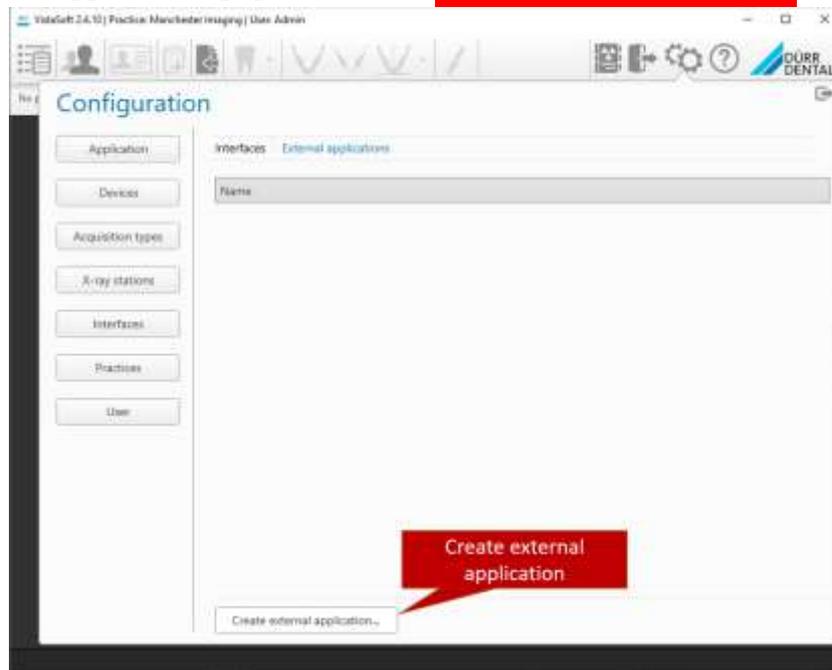


Select 'Third-party application' and complete the form as shown below using Ctrl-v to paste into the boxes:

Name: **AssistDent**

Path (N.B. By default setting to use is): **C:\Program Files (x86)\Manchester Imaging Ltd\AssistDent\AssistDent.exe**

Parameter: **--image "%PathToFile%" --title "%PatientId% - %PatientFamilyName%,%PatientGivenName%"**

Once the information is entered hit the top-right button to **Save and Exit**.

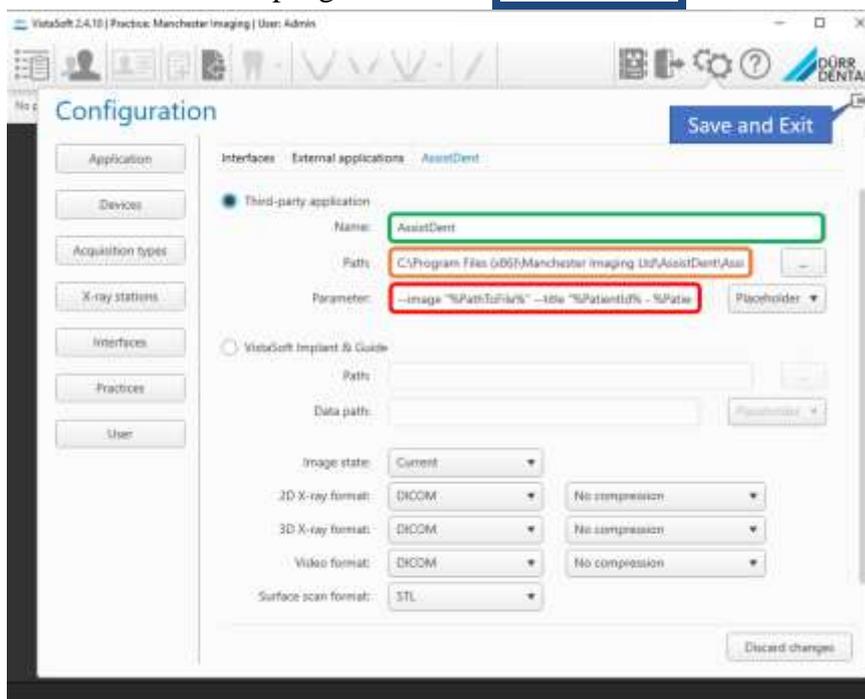

Run AssistDent from VistaSoft

Within VistaSoft, select a bitewing radiograph and then click the **Export to external application** button

