

Getting Started with AssistDent®


This document applies to AssistDent v1.7.

Once downloaded, follow the steps below to install AssistDent and start detecting caries


Install AssistDent

Double click the downloaded file, AssistDent.exe.

The AssistDent installer starts:

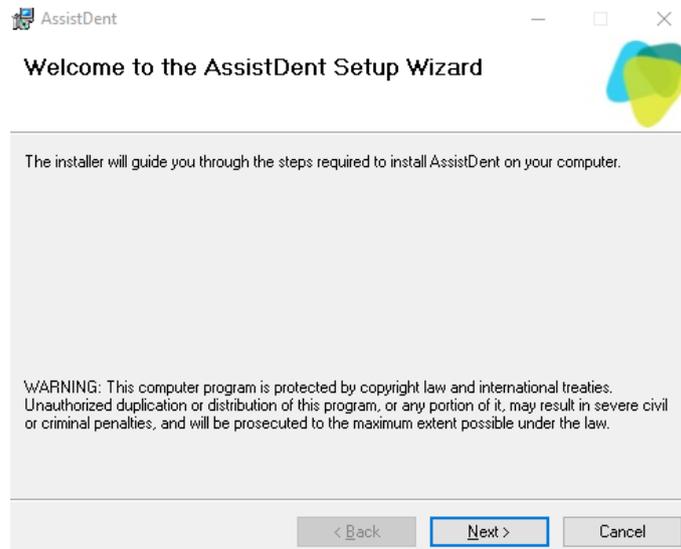

Follow the steps in the wizard. When successful the following message is displayed:

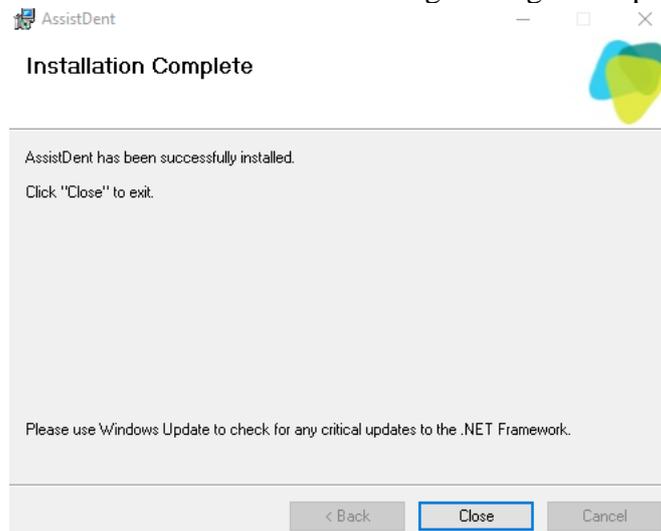

Start AssistDent

Start AssistDent from the Windows Start Menu or the shortcut on your desktop.



Register

AssistDent starts. Please register:




The registration form is titled "AssistDent®" and "Register to start your free trial of AssistDent". It contains the following fields: Name, Email, Practice/Company, Contact Phone, Address Line 1, Address Line 2, City, Postal Code, and Country. A "Register" button is located at the bottom right. Below the form, there is a disclaimer: "By registering you are accepting the [terms and conditions](#) of using the software. Free trial period: Your free-trial period starts immediately after sign up. At the end of your trial, you have the option to begin your subscription. Your eligibility and access to use an AssistDent free trial offer is determined solely by Manchester Imaging Limited."


Use AssistDent

After registering, browse for a bitewing image to analyze with AssistDent.

Alternatively, you can choose a bitewing image file on your computer and select 'Open With AssistDent'

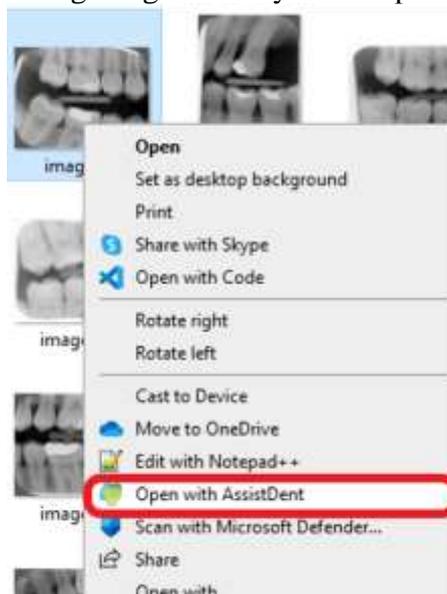

The selected bitewing radiograph is displayed, and caries detection performed:

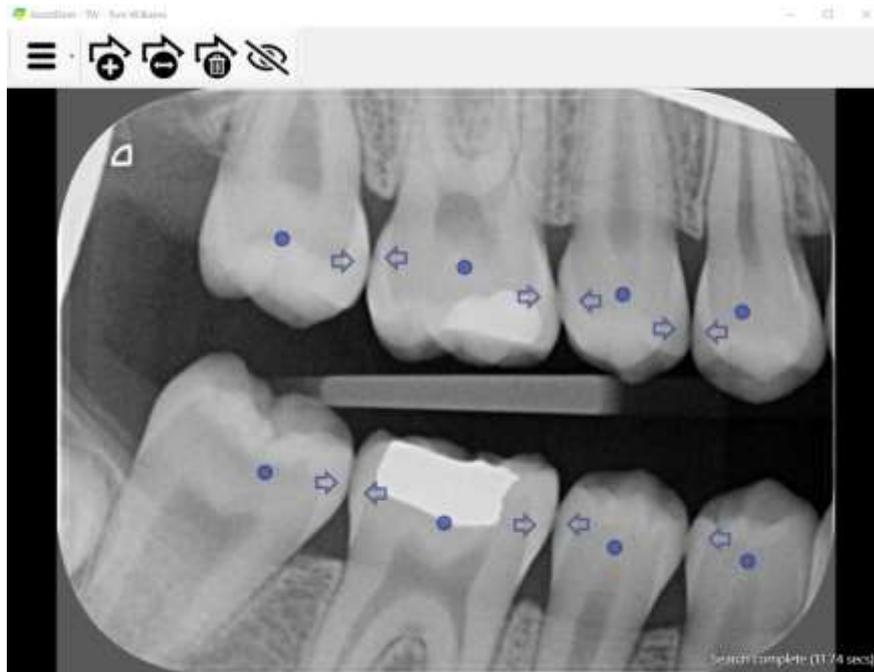

AssistDent User Guide

Access the User Guide from the main menu for instructions on how to use AssistDent

