

Configure AssistDent® with EXAMINE^{PRO}

(Includes AssistDent 'Automatic Assist' Feature)

AssistDent has an 'Automatic Assist' feature which automatically analyses all selected radiographs placed in the specified folder, plus an additional '**Automatic Export**' feature which automatically saves the analysed images. This guide details how these features can be used to facilitate integration with Examine Pro software.


N.B. This document applies to AssistDent v1.7 onwards and Examine Pro® v8.19 and earlier.

Setup AssistDent Import and Export Folders

STEP 1: In your 'Documents' folder, create two new folders

- (a) 'To AssistDent' to hold/manage/send images that will be imported into AssistDent
- (b) 'From AssistDent' to hold/manage/receive images exported from AssistDent

Tip: Make a note of where these folders have been setup on your Surgery P.C.

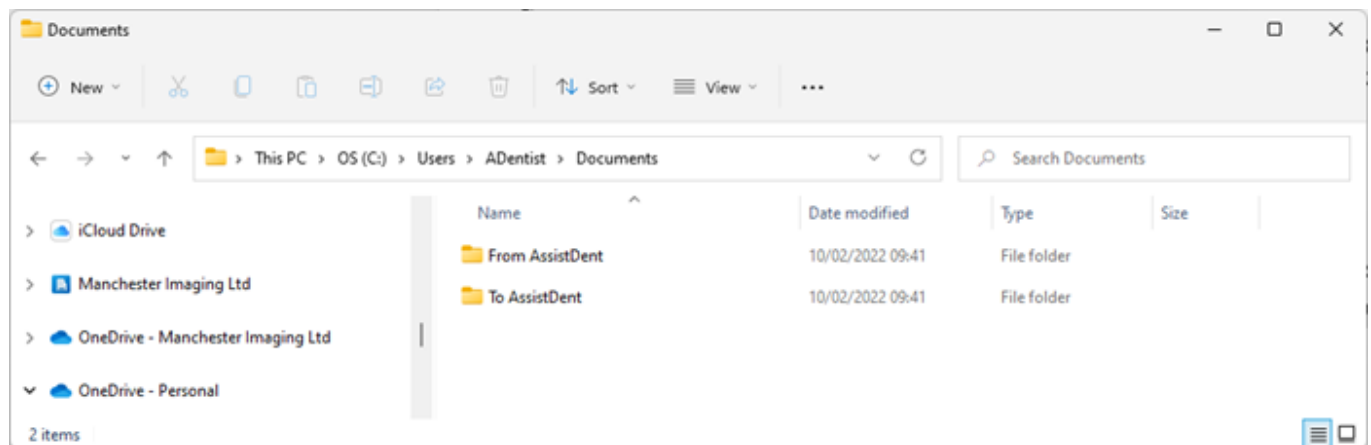

Setup AssistDent for 'Automatic Assist'




STEP 2: Click the 'up' arrow ^ to 'show hidden icons'

to locate the in-tray AssistDent icon in the windows taskbar

STEP 3: Right click on the AssistDent icon and choose 'Settings'

STEP 4: Tick the 'Enable Automatic Assist' box

STEP 5: Type the full pathway for the newly created folder; 'To AssistDent'

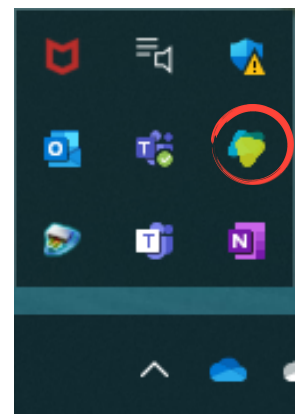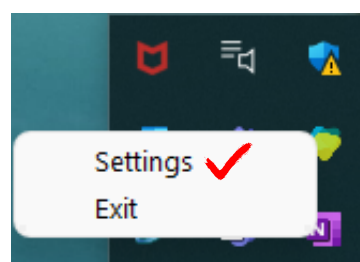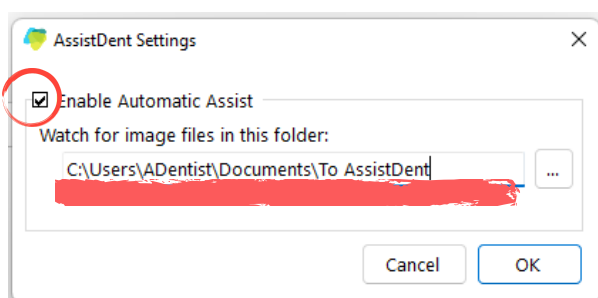




Setup AssistDent for 'Automatic Export'

STEP 1: Open AssistDent

STEP 2: Click on the 'Main Menu' Tab  then select 'Settings'


Open AssistDent and Click on Settings

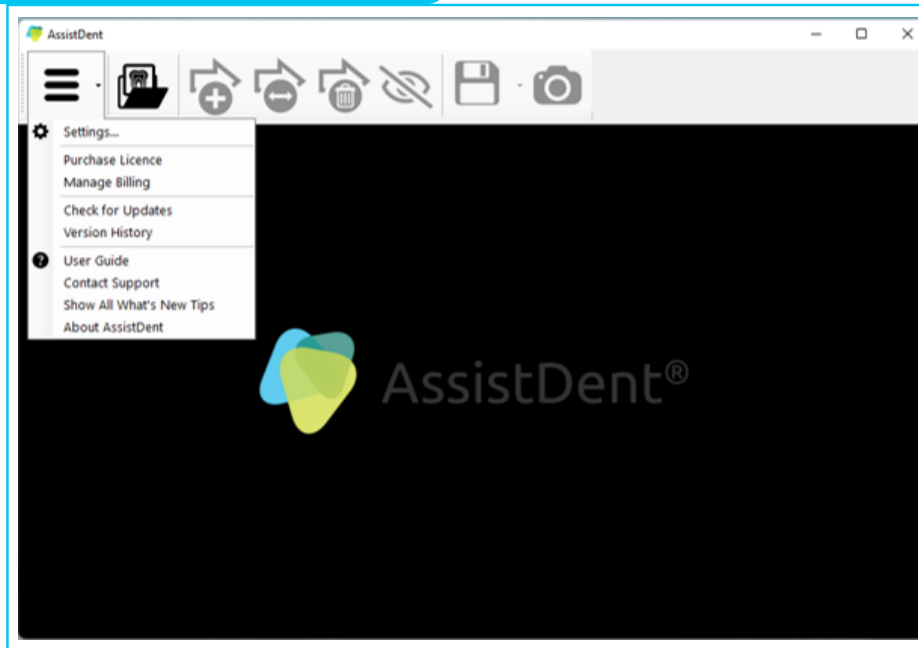

STEP 3: Tick the 'Automatically export annotated image on exit' box

STEP 4: Tick the 'Include teeth indicators in exported image' box

Settings Instructions

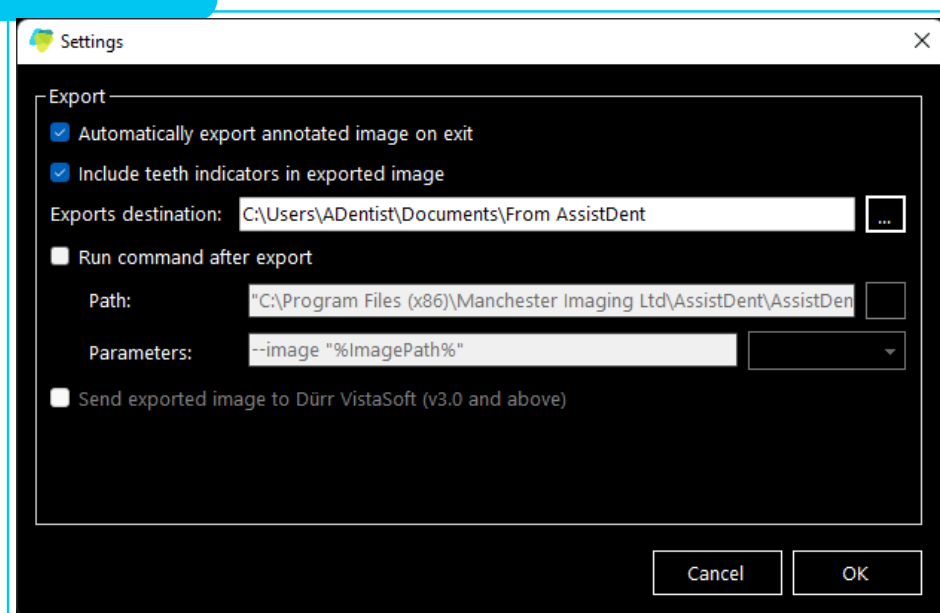

STEP 5: Type in the 'Export destination' pathway to the 'From AssistDent' folder (already created earlier in the setup)




Export Images from EXAMINE^{PRO} into AssistDent for Analysis


STEP 1: Start (or open) **EXAMINE^{PRO}**

STEP 2: Select the Bitewing Radiograph to be analysed (high resolution required)


Export Radiograph from EXAMINE^{PRO}

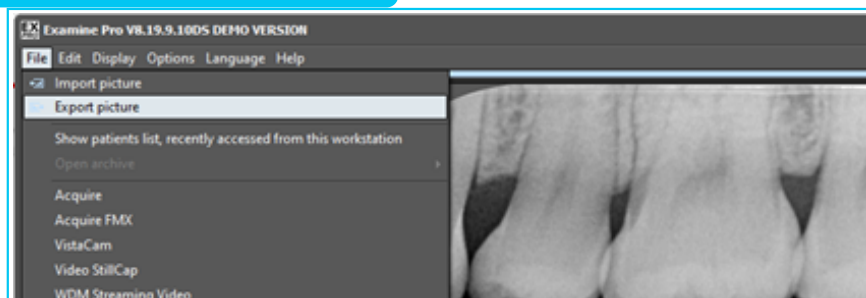

STEP 3: Click File then 'Export Picture' and choose the 'To AssistDent' folder created earlier and for continual best quality resolution to 'Save As Type' 'Tiff 24-bit Colour'

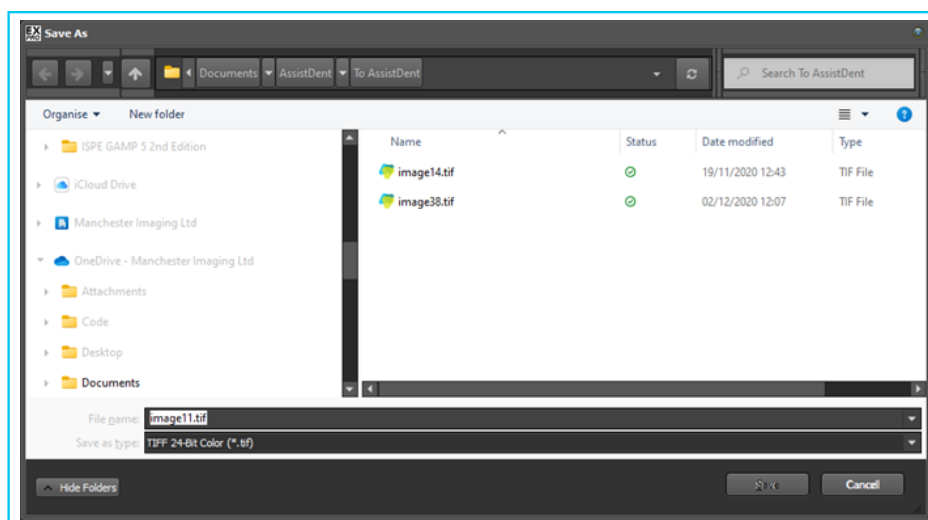

STEP 4: AssistDent automatically starts and analysis commences

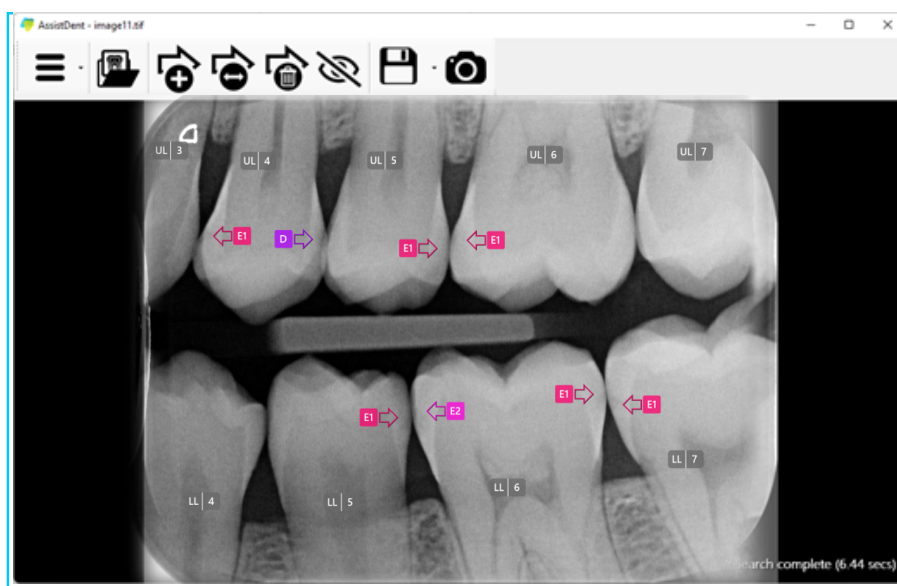





Import Analysis Results from AssistDent into EXAMINE^{PRO}

STEP 1: Close AssistDent upon completion of the analysis. The results are automatically saved in the 'From AssistDent' folder, as previously configured.

STEP 2: In EXAMINE^{PRO} select File, then click on 'Import Picture'

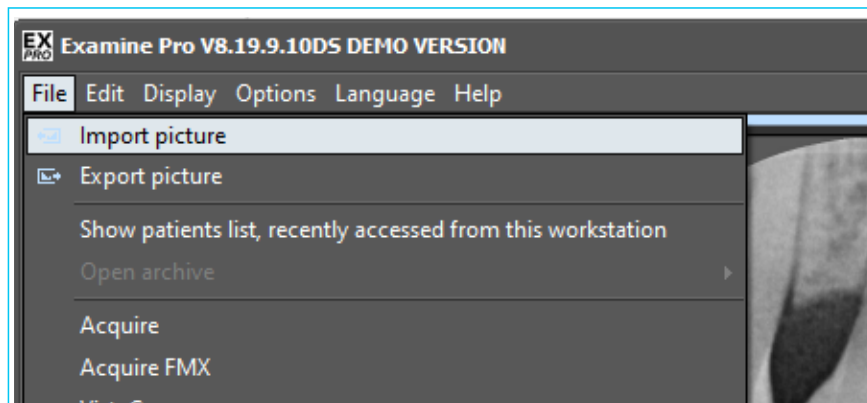

STEP 3: Navigate to the 'From AssistDent' folder, double click on the recently exported image to select it and then click on the 'Import' button

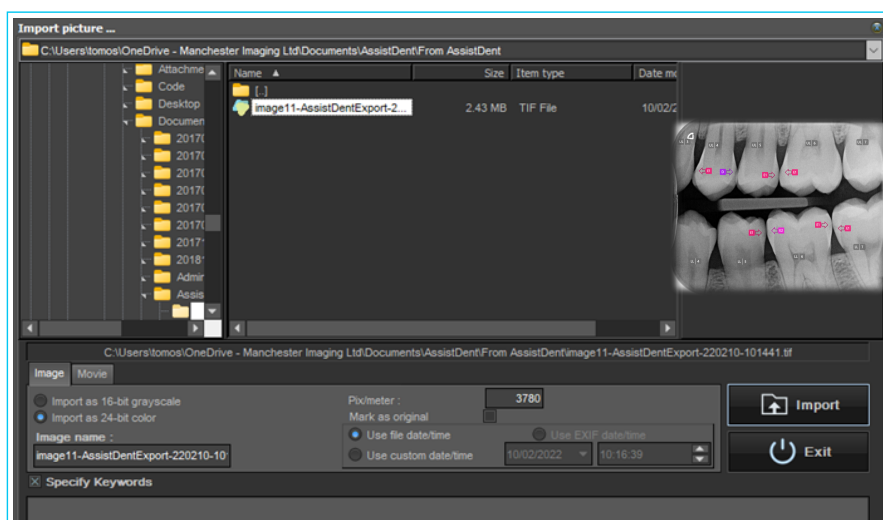

STEP 4: The AssistDent analysed image is then filed in the patient records

