



Configure AssistDent® with VistaSoft®




(A Plugin to Interface with Dürr Dental's VistaSoft)

N.B. This document applies to AssistDent v1.7 and above plus VistaSoft v3.0 and above


STEP 1: Open VistaSoft, click the '**Configuration**' button 

STEP 2: Click on '**Features**' 

STEP 3: At the bottom of the page select the '**Add External Application**' button 

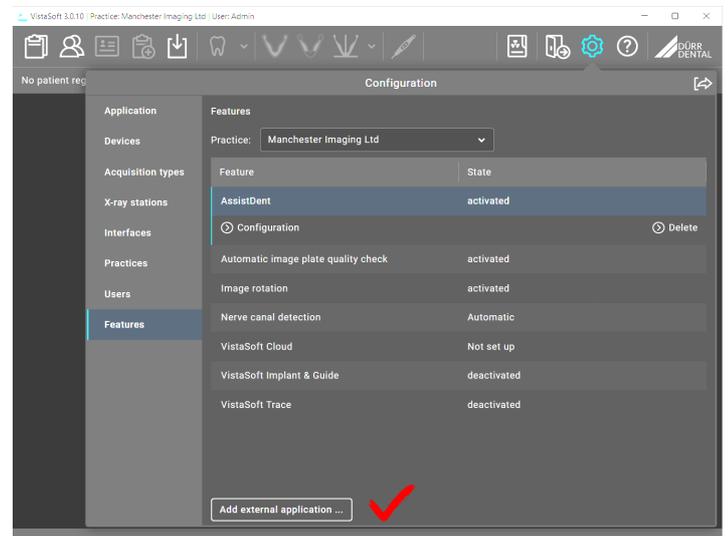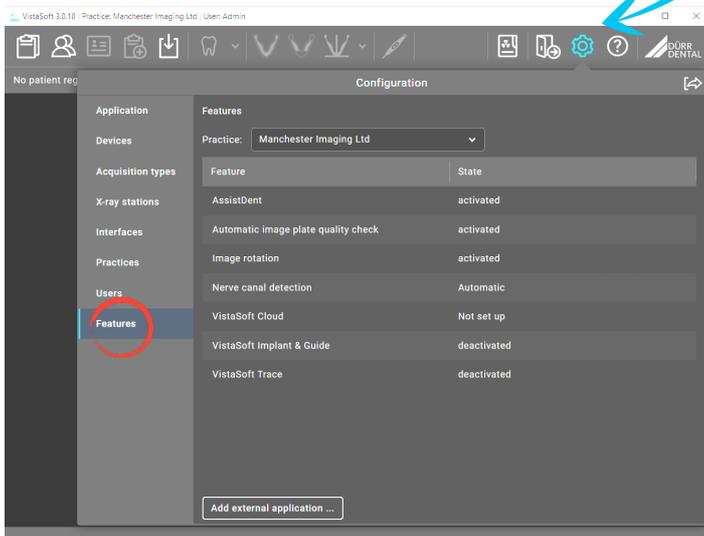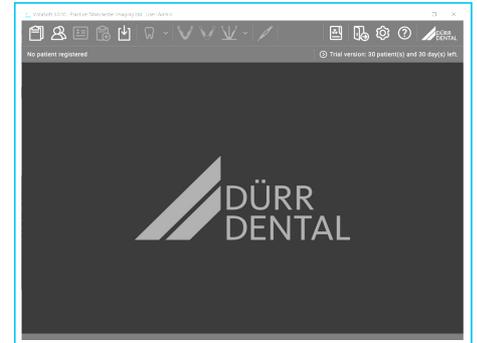

STEP 5: Fill in the following fields; 'External application name' - 'Executable path' - 'Parameter'

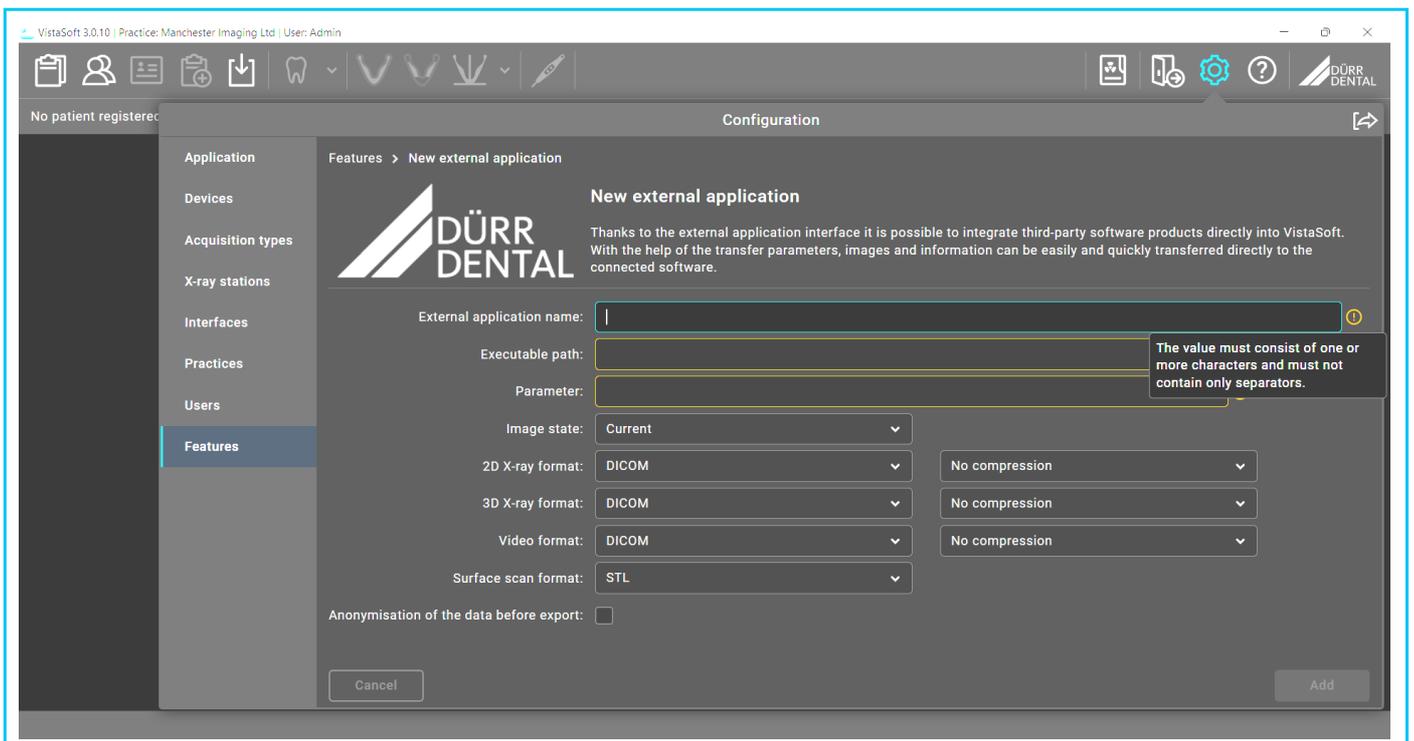



New External Application Settings



STEP 6: Use the following details to copy and paste direct into the box...

External application name: AssistDent

Executable path: C:\Program Files\Manchester Imaging Ltd\AssistDent\AssistDent.exe

Parameter: --image "%PathToFile%" --title "%PatientId% -%PatientFamilyName%,%PatientGivenName%" --sourceId "%ImageUid%"

STEP 7: Once the information is entered to complete click on Save

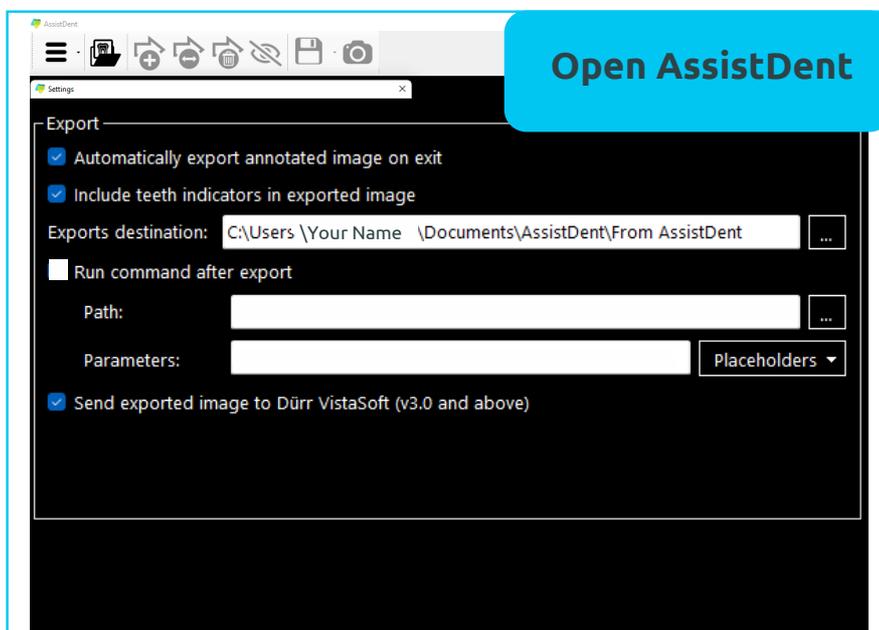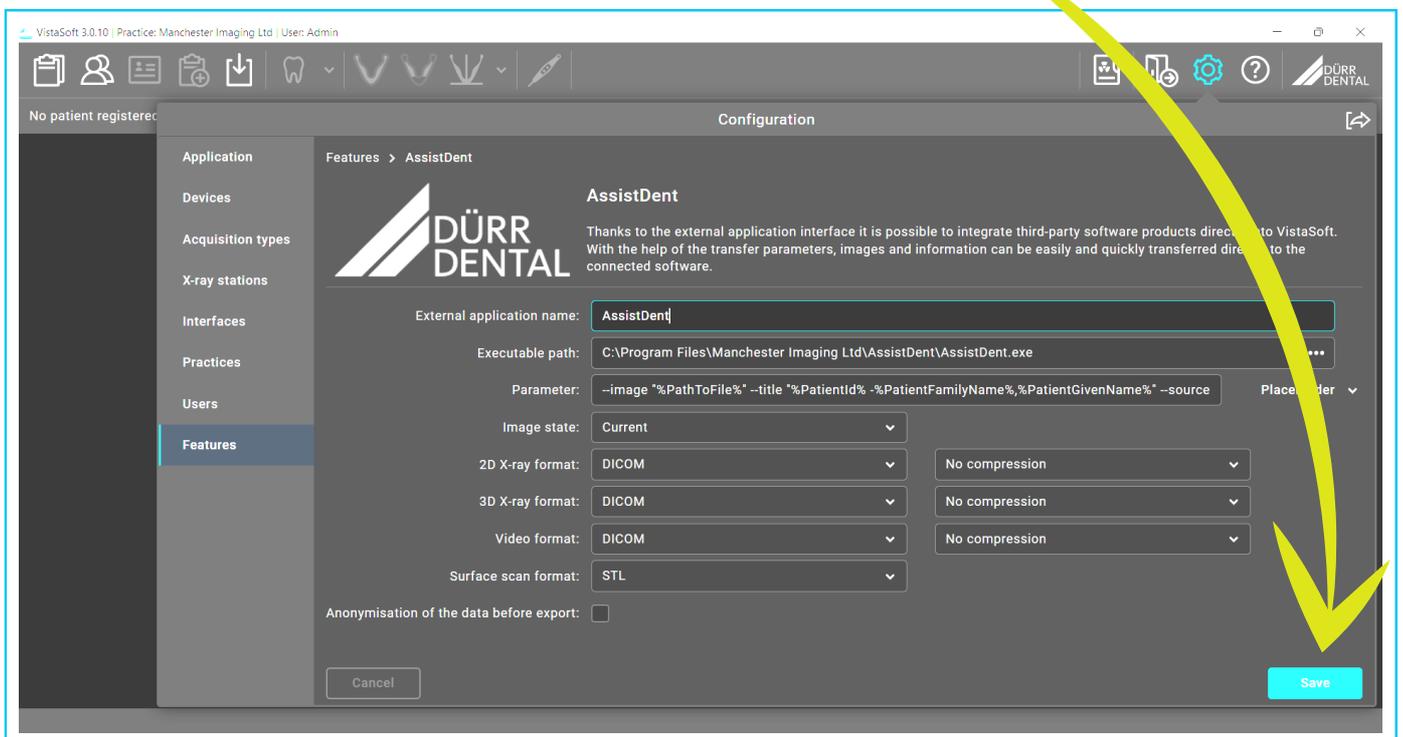

STEP 8: Go to Menu  and click on **'Settings'**

STEP 9: Ensure that the following boxes are all ticked...

- Automatically export annotated image on exit
- Include teeth indicators in exported image
- Run command after export
- Path:
- Parameters:
- Send exported image to Dürr VistaSoft (v3.0 and above)

STEP 10: Click OK to save and close AssistDent





Export to External Application

STEP 11: Open VistaSoft and select a Bitewing Radiograph

STEP 12: Then click the *'Export image and start external application'* button




STEP 13: AssistDent automatically opens in the background and starts the analysis

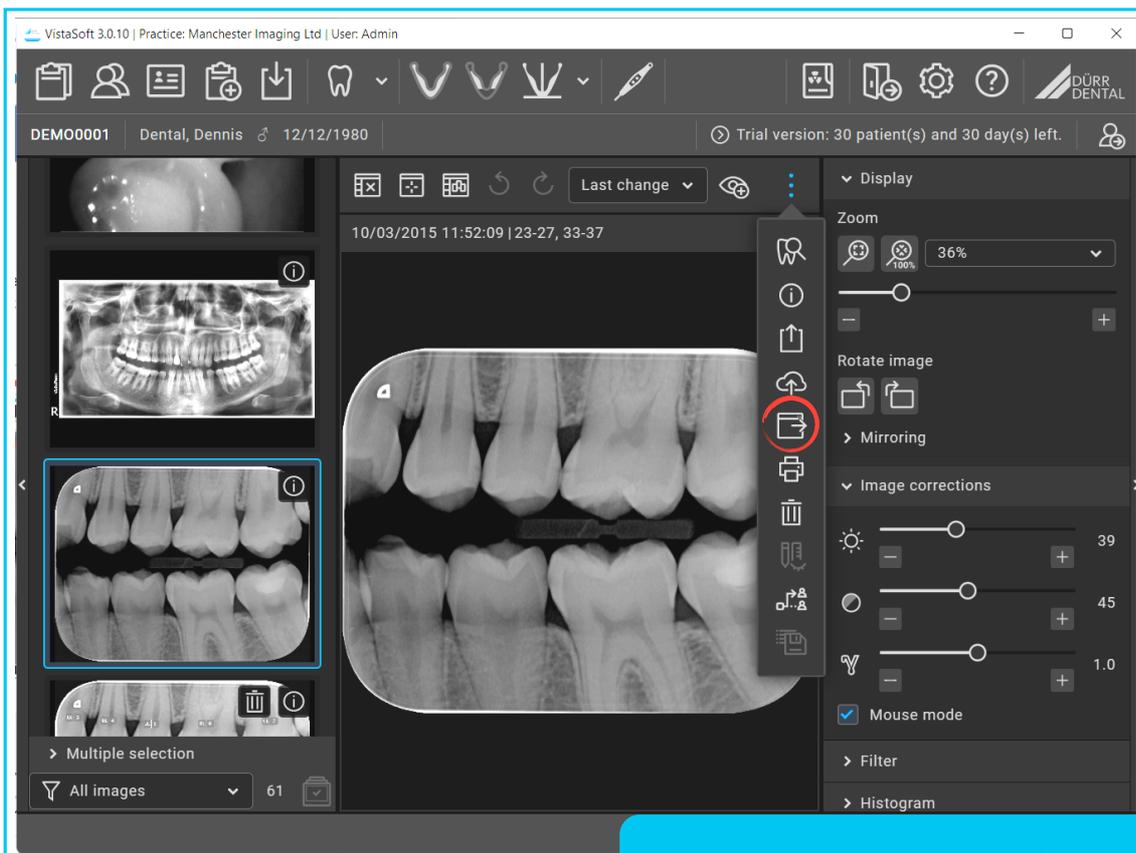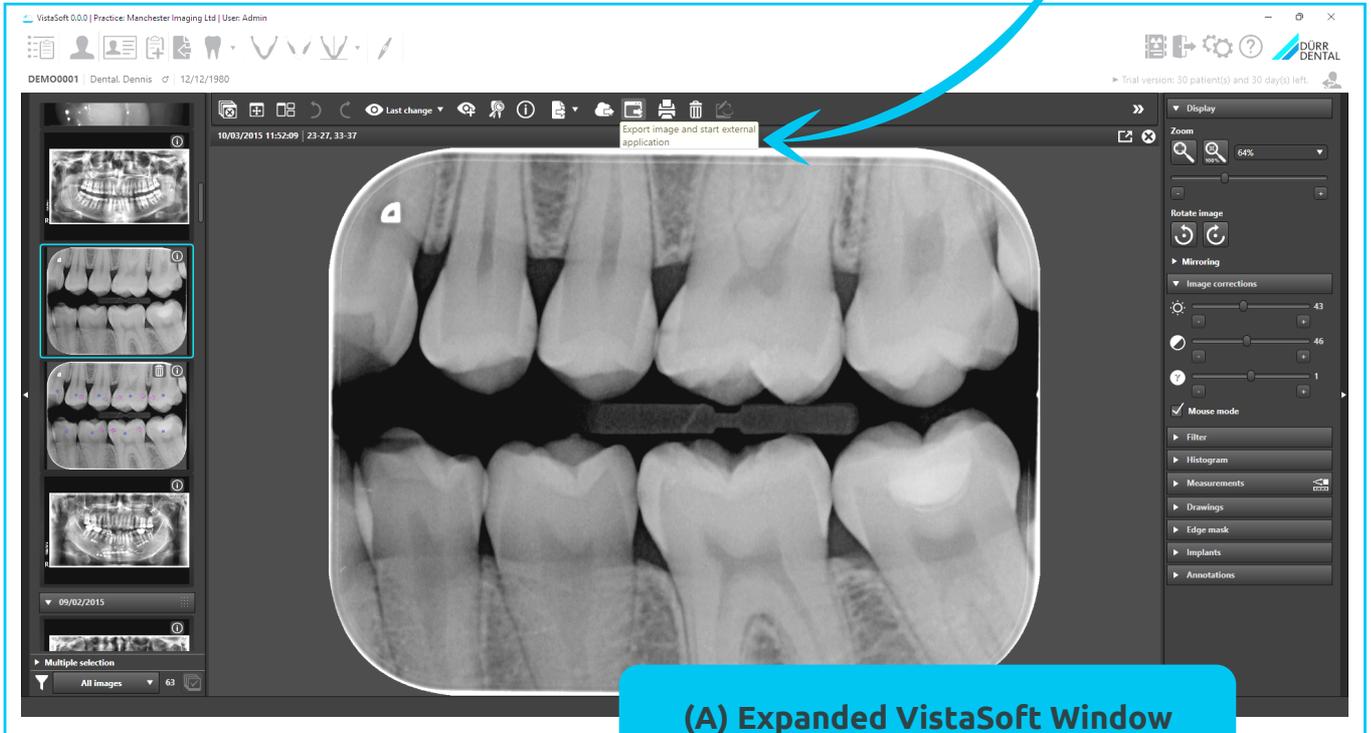



AssistDent Opens and Analysis Automatically Commences

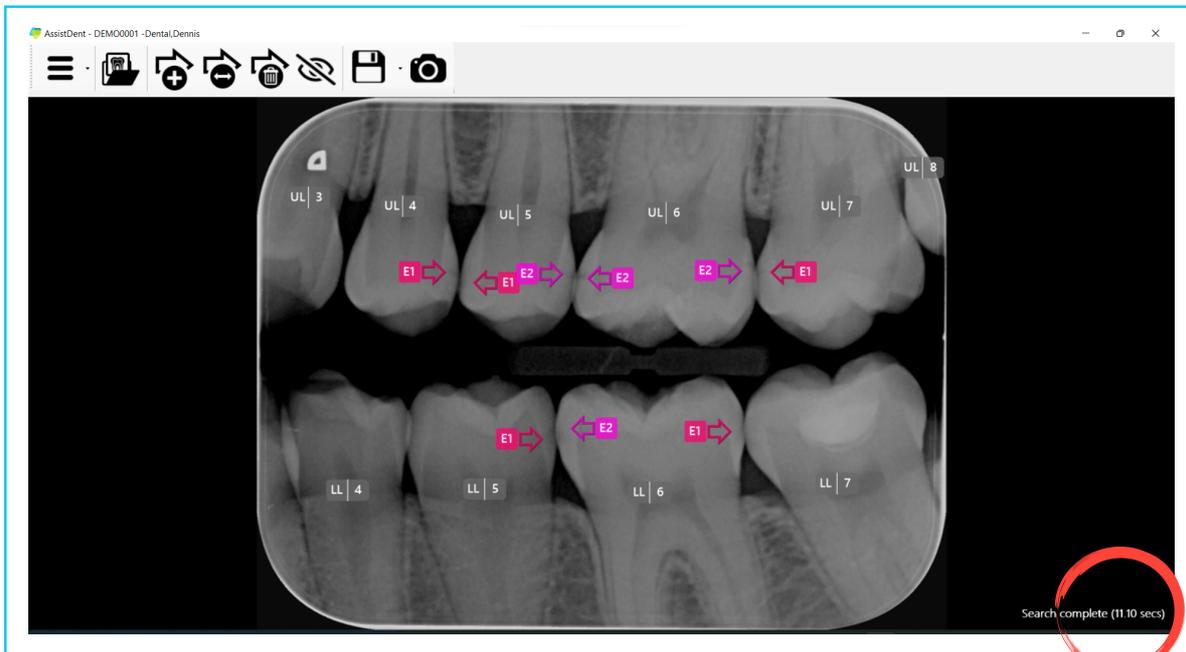

Running AssistDent from VistaSoft

Import of Analysed Bitewing Radiograph

Upon completion of analysis, the resulting newly labelled radiograph is imported straight into the Patient's folder. **N.B.** Whilst the original radiograph cannot be deleted, it is possible to delete the AssistDent analysed image if necessary, simply by highlighting it and then clicking on the delete button at the top of the screen.

